NEW

大师椅 Master
SMART MASSAGE CHAIR

OGAWA

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www.ogawaworld.net.cn

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Contents

| | |
|----------------------|----|
| Safety Instructions | 26 |
| Warranty Claims | 30 |
| Parts of the Chair | 31 |
| Moving the Chair | 33 |
| Controller Diagram | 35 |
| Operation Guides | 36 |
| Features | 49 |
| Chair Specifications | 51 |
| Troubleshooting | 52 |
| Product Maintenances | 53 |

01 Safety instructions

Warning!

The plug must be fully inserted into an appropriate outlet. Improper connection of the equipment-grounding conductor can result in an electric shock.

Check with a qualified electrician and servicemen if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided by the product. If the outlet is unsuitable, please ensure that a proper outlet is installed by qualified electrician.

When using the massage chair, basic precautions should always be followed to reduce the risk of electric shock, an injury or burns to the person.

Read the following details before usage:

1. Be sure to insert massage chair plug into the proper electrical outlet.
2. Make sure the power plug is securely inserted into the proper electrical outlet to reduce the risk of short-circuit and fire.
3. Always unplug the massage chair from the electrical outlet immediately after use and before cleaning.

4. DO NOT use the massage chair in places where the humidity is high.
5. DO NOT use the massage chair outdoors and avoid places with direct sunlight.
6. If the power goes off during use, be sure to turn the power switch to the OFF position and pull the power plug out from the socket to prevent any damages at the time when the power comes back.
7. Never operate the massage chair if the power cord is wet or damaged.
8. Never plug/unplug the power cord from the socket with wet hands.
9. Close and continuous supervision is necessary when this appliance is used by, on, or near children, invalids or disabled persons.
10. Use this appliance only for its intended use as described in this user manual.
11. Do not use any attachments other than those recommended by OGAWA.
12. Do not stand, sit or jump in the calves rest when massage chair is usage. This may result in an accident an injury.
13. Do not attempt to stand on the backrest and armrest .This may result in an accident and injury.
14. Be careful not to fall asleep during the massage.

15. Always keep the massage chair clean and never insert any object into the gaps on and around the massage chair.
16. Do not remove the massage chair by pulling the power supply cord.
17. Be sure to check that there is no one (children in particular), pet or obstacles under the massage chair prior to lowering the leg rest /backrest.
18. Always check whether the surface of the massage chair is torn or ripped before usage. If you find any unfamiliar rips in any area of the cloth in the backrest, do not use the massage chair. Kindly contact OGAWA service center immediately for repairs.
19. This appliance is designed for household use only.
20. For those suffering from illnesses, please consult your doctor on whether you can use the massage chair if in doubt.
21. The following persons are requested to consult a doctor prior to use in order to reduce the risk of user feeling sick or suffering from injury:
 - Those who have malignant tumor
 - Those who suffer from heart disease
 - Those who are pregnant experiencing menstruation and puerperal fever
 - Those who require complete rest
 - Those who are under medical treatment

- Those who suffer from back problems or whose backbone is abnormal or bent
- 22.If you experience pain during usage, stop usage immediately.
If any injuries occurs, consult a doctor immediately
 - 23.Never massage on swollen or inflamed parts of the body.
 - 24.This product is non-professional appliance designed to provide a comfortable massage. Do not use this product as a substitute for medical treatment.
 - 25.Do not use this massage chair for more than 30minutes at one time and kindly limit the massage to a concentrated area for not more than 15minutes. Over-massaging can product overheating thus giving it a shorter lifespan.
 - 26.Do not operate where aerosol (spray) products are used or where oxygen is being administered.
 - 27.The massage chair is not suitable for children aged 12years old and below.
 - 28.If the supply is damaged, it must be replaced by a special cord or assembly available from the manufacturer or its service agent.
 - 29.This product is equipped with heating function, DO NOT use over insensitive skin area or in the presence of poor circulation.


02 Warranty Claims

1. Warranty is not transferable.
2. This warranty is effective only if the product is purchased from OGAWA.
3. Proof of purchase (original receipt) is required for all warranty repairs.
4. All implied warranties, including but not limited to these implied warranties of fitness and merchantability, are limited in the total duration of 1 year from the original purchase date.
5. Damage due to misuse, improper treatment and unauthorized modifications and repairs are not covered by this warranty.
6. Warranty does not cover accessories and attachments which do not belong to the massage chair.
7. Warranty is not effective to rental, Business, commercial, institutional, or other non-residential users.
8. All services covered by this warranty must be approved by OGAWA and repaired by authorized technicians only.
9. If replacement parts for defective materials are not available, OGAWA reserves the right to make substitution in lieu of repair or replacement.

05 Moving The Chair

1. Restore the footrest to its original position.
2. Recline the chair to tilt approximately 45degrees. Move the chair using the wheels as the fulcrum.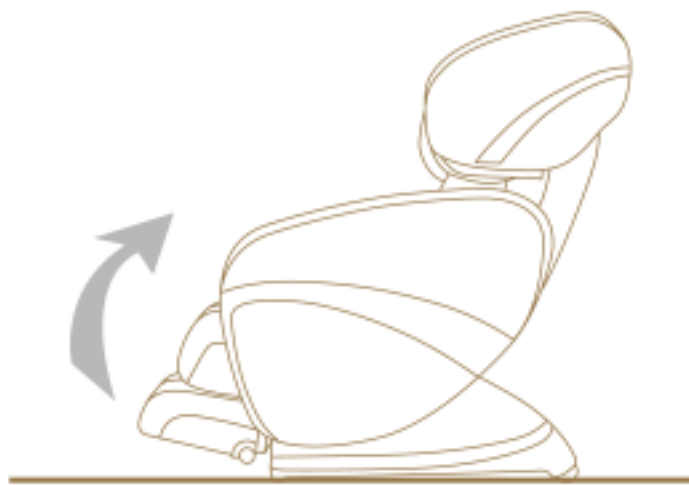


Note:



Always return the footrest to its original length and restore it to its original position before moving the chair, otherwise injury during movement may result.

Do not move the chair while a person is seated on it.

Place a mat on the floor and gently roll the chair over it to avoid damaging the floor.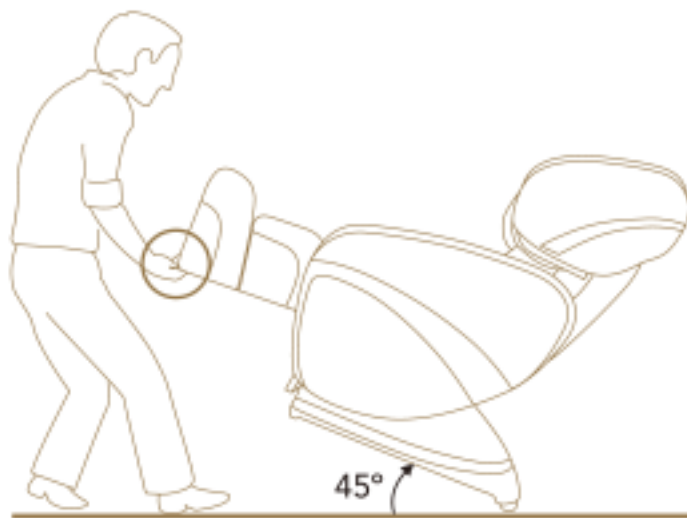

03 Parts of the Chair

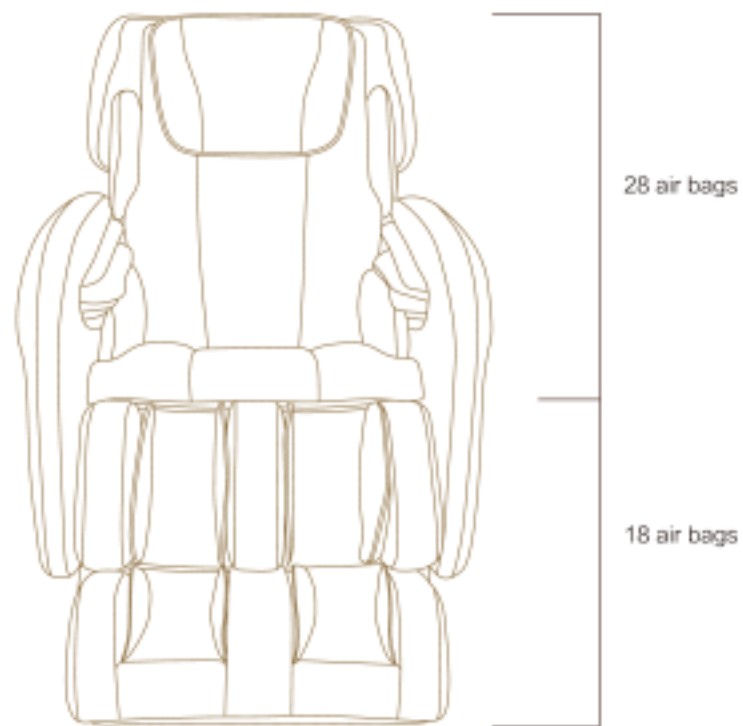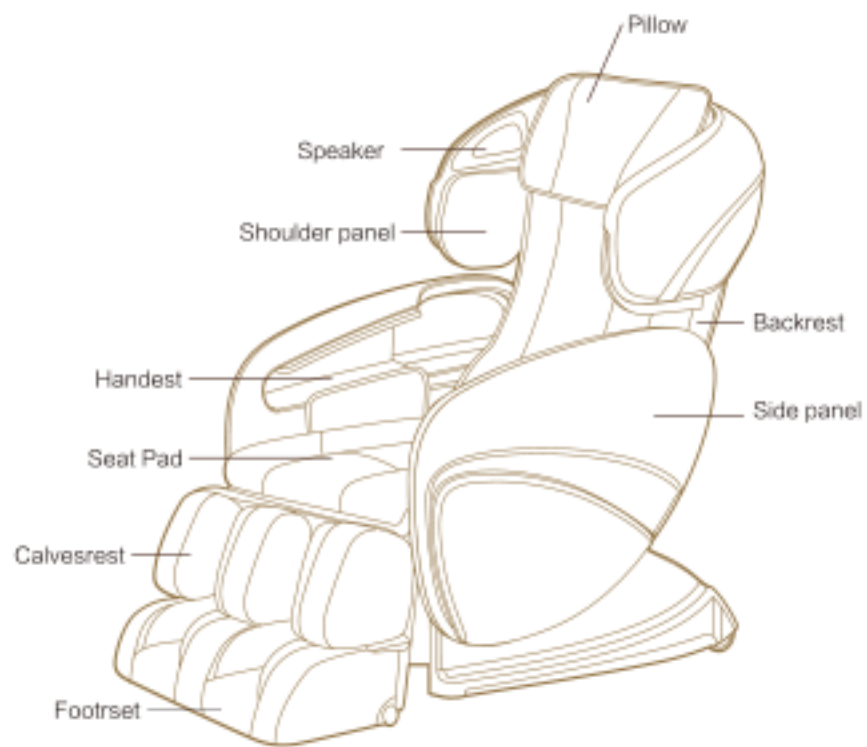06 Controller Diagram

07 Operation Guides

Connect to the power supply, and turn the switch to "I" position, chair is now in idle mode. During its idle mode, ON/OFF button will blink, which means the electric is connected. During its idle mode, only ON/OFF, Pause, Incline, Recline, Incline Leg and Recline Leg button can be operated.

Control Buttons for the Massage Chair

1) ON/OFF

The Start/Stop Control Button

Press this button to light ON the LCD screen. Press any massage function button to start massage. This will initiate the timer function. Press this button again to stop all massage functions, massage chair will return to its original position.

2)TIMER

Time interval Control Button

When a massage function is activated, the timer is preset at 15 minutes. Every single press on this button will change the timer interval in the sequence of "20>25>30>5>10>15>20" minutes.

Note: RECOVERY auto massage program is fixed 20 minutes.

3)PAUSE

Press this button once to halt running massage function and countdown timer. During pause mode, the pause button will be blinking. Press this button again to cease the "Pause" function and resume previous massage function as well as countdown timer.

4)ZERO-G

Press this button to active the Zero-G1 function in default, press the button again to activate Zero-G2 function, press again to return to idle position.

Note: If this button is pressed while moving to Zero-G1, massage chair will stop at latest angle. If Zero-G button is not press while moving to Zero-G1, massage will stop at preset Zero-G1 angle. Same conditions apply to Zero-G2 function.

Note:1. ZERO-G button is NOT controllable in below massage programs:

THAI

RECOVERY

QUICK MASSAGE

2.zero gravity function can only adjust to the state of zero gravity

1:ELDER

5)INCLINE LEG / RECLINE LEG

Angle Adjustment Control Button for Footrest

These two control buttons can adjust massage angle of the Footrest. Press on the Incline Leg or Recline Leg control button to adjust Footrest to a desire comfortable position. Press either button again to stop it.

6)INCLINE/RECLINE

Angle Adjustment Control Button for Massage Chair

These two control buttons can adjust the Incline/Recline angle of the massage chair. Press on the Incline or Recline control button to adjust the Backrest and Footrest to a desire comfortable position. Press either button again to stop it.

The following are the Auto Programs available:

7)GENTLEMAN

8)LADY

9)ELDER *1

10)QUICK MASSAGE*2*4

11)THAI*2

12)RECOVERY*2*3

13) DEEP TISSUE*5

14)CHINESE

15)DEEP SHOULDER

16)DEEP WAIST

17)FULL BACK

*1 Only Zero-G1 is available for adjustment in particular massage program

*2 ZeroG button is NOT controllable in particular massage program

*3 Massage time is fixed at 20 minute. After 20 minute, massage chair will automatically shut down at ZeroG position

*4 The automatic massage program timing for 10 minutes. 10 minutes later, the massage chair will be automatically reset to turn it off

*5 The automatic massage program pose for a handstand Angle, you are unwell should Angle users please careful!

18)JOYSTICK

Upward/Downward adjustment

The position adjustment button for spot and partial massage

When the spot or partial massage functions are activated, joystick can be used to move the massage rollers. Hold the joystick Up/Down to move the massage rollers upward/downward respectively. Release the joystick when the desired position is reached.

When auto Backrest Scan can't detect desire massage stroke,

push Joystick Upward/Downward to perform fine adjustment within 10 second after buzzer sounded. Release joystick when desire position is reached.

The 3D adjustment

Push the joystick Left/Right to adjust the 3D massage roller forward/backward respectively.

There are 6 different levels available for adjustments to suite various massage strength requirement.

19)TAPPING

Tapping function Start/Stop Control Button

Press this button to activate Tapping function, press this button again to stop the Tapping function.


Note:During Auto Program mode, pressing this button can disable/enable the Tapping function.

20)ZIG – ZAG

ZIG – ZAG function Start/Stop Control Button

Press this button to activate evolutionary full body twisting massage function with airbags. Press this button again to stop ZIG – ZAG function.

21)FOOT ROLLING

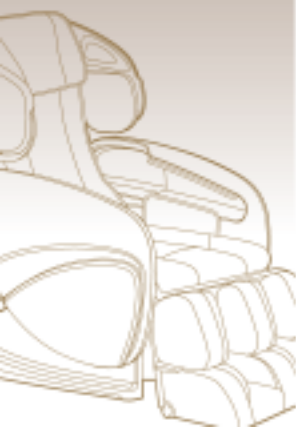levels of intensity to be adjusted

Press this button once to activate rhythmic full body air pressure massage. Press this button again to stop the function.

Note: During Auto Program mode, pressing this button can disable/enable the air function.

When either one of the Auto Massage or Scrapping Program (except Full Air) is selected, massage chair will automatically perform a Backrest Scanning to detect shoulder position on user for most comfortable massage experience.

Buzzer will sound to remind users when countdown timer less than 1 minute.

Finish Operating

Please press the "ON/OFF" button after use. Massage Rollers will restore to its original position; then switch the power supply button under the chair to "O" and unplug the machine from the wire socket.

08 Features

OGAWA massage Chair is controlled by a microcomputer that performs various intelligent massage type and combinations such as Kneading, Swedish, Tapping, Shiatsu, Clapping, Rolling, Scrapping, Foot Roller, Vibration and Air Massage. All these types performing massage with accurate on the acupressure points, to effectively alleviate fatigue, promote metabolism and improve blood circulation. Equipped with 8 set of full body automatic massage programs thoroughly fulfill your various need.

Provides 8 set of Auto massage programs and various type of manual massage function provides to various requirement in publics.

In addition to precise points massage, 3D technology massage that is specially designed for the back according to the curve of the human body to provide a more effective and thorough massage.

- 1.The massage rollers use an interlocking gear to provide an in-depth massage to the acupressure points on the back.
- 2.Designed with double automatic adjustment, the Chair's Backrest can be adjusted from 115 degree to 160 degree, while the Footrest is from 0 degree to 90 degree.
- 3.The Timer Function can be adjusted according to your desired

Foot roll function Start/Stop Control Button

Press this button once to activate rhythmic foot base rolling function, press this button again to stop the foot base rolling function.

Note: During Auto Program mode, pressing this button can disable/enable the Foot Rolling function.

22)CRAPPING

Scraping function Start/Stop Control Button

Press this button to activate Scraping function, press this button again to stop the Scraping function.

23)HEAT

Heat function Start/Stop Control Button

Press this button to activate knees heat function, press this button again to stop the Heat function.

Note: 1.Calvesrest air bags must be turned

During Auto Program mode, pressing this button can disable/enable the Heat function.

Manual Massage Program

24)KNEADING

Kneading function Start/Stop Control Button - 5 levels

of strength to be adjusted

Press this button to activate rhythmic mode 1 Kneading function for a full back massage at medium strength. Press the button again to activate mode 2 Kneading function. Press the button for the third time to stop Kneading function and switch to mode 1 Swedish function.

24)SWEDISH

Swedish function Start/Stop Control Button - 5 levels of speed to be adjusted.

When mode 2 Kneading function is engage, press this button to activate rhythmic mode 1 Swedish function for a full back massage at medium speed. Press this button again to activate mode 2 Swedish function. Press once more to stop the Swedish function. To activate Swedish function directly, press this button 3 times during idle mode.

25)TAPPING

Tapping function Start/Stop Control Button - 3 levels of width, 5 levels of speed to be adjusted.

Press this button to activate rhythmic mode 1 Tapping function for a full back massage at medium speed. Press the button again to activate mode 2 Tapping function.

Press the button for the third time to stop Tapping function and

duration of the massage programmer.

4. Liquid Crystal Display for a user friendly operation even in the dark.
5. Designed with power surge protector as well as automatic thermal sensor and auto shutdown for safety.
6. The operation is controlled by intelligent microprocessor for excellent performance and consistent quality.
7. Ergonomically designed to enhance massage experience.

09 Chair Specifications

| | |
|------------------------------------|--|
| Product Name | Ogawa Smart DeLight |
| Model no. | OG7558G |
| Power source | 220V~ |
| Alternating current | 50Hz |
| Power consumption | 300W |
| Power of speakers | 14W (2 sides) |
| Timer | 5/10/15/20/25/30 minutes |
| Maximum operating period | 30 minutes |
| Dimension (Length/width/height) | Upright position: 136x 87x 129 cm |
| | Reclined position: 182x 87x 96 cm |
| Weight | Product gross weight: 140 kg |
| | Product net weight: 122 kg |
| Length of the wire | Controller wire: Approx 1.5M |
| | Power supply wire: Approx 1.8M |
| | Music connection wire: Approx 15+75CM |
| Usage condition | Environment temperature: 10°C ~40°C |
| | Contrasting humidity: 30~85RH |
| Storage condition | Storage temperature: 20°C ~60°C |
| | Storage humidity level : 30~85RH |
| Safety feature | Equipped with overheated and power surge safety protection |
| Using benefits | Increasing blood circulation Relieving muscular fatigue |

switch to mode 1 Shiatsu function.

25)SHIATSU

Shiatsu function Start/Stop Control Button - 3 levels of width, 5 levels of speed to be adjusted.

When mode 2 Tapping function is engage, press this button to activate rhythmic mode 1 Shiatsu function for a full back massage at medium speed. Press this button again to activate mode 2 Shiatsu function. Press once more to stop the Shiatsu function. To activate Shiatsu function directly, press this button 3 times during idle mode.

26)CLAPPING

Clapping function Start/Stop Control Button - 3 levels of width, 5 levels of speed to be adjusted

Press this button to activate rhythmic mode 1 Clapping function for a full back massage at medium speed. Press the button again to activate mode 2 Clapping function.

Press the button for the third time to stop Clapping function and switch to Rolling function.

26) ROLLING

Rolling function Start/Stop Control Button - 3 levels of width to be adjusted

When mode 2 Clapping function is engage, press this button to activate Rolling function for a full back massage. Press this button again to stop the Rolling function. To activate Rolling directly, press this button 3 times during idle mode.

Note: This function cannot perform "Spot" massage function.

27)SPOT

The Spot Back massage

Press this button to activate Spot function, press this button again to stop the Spot function.

28)WIDTH

The Width Adjustment Button for Tapping, Clapping, Shiatsu and Rolling function

When Tapping, Clapping, Shiatsu or Rolling function is engage, every single press on this button will change the massage width interval in the sequence of "Narrow > Medium > Wide" in cycle.

29)PARTIAL

The Partial Back massage

Press this button to activate Partial function, press this button again to stop the Partial function.

Note: Partial Massage will take the latest position as the center to perform Partial Massage with the distance approx. 8 cm

10 Troubleshooting

This massage chair is controlled by microcomputer and is susceptible to power surge even by +10% or -15%. The massage chair will show the following strange circumstances:

| Situations | Solution |
|---|--|
| The Massage Chair cannot operate even though it has been connected to the power supply. | <ol style="list-style-type: none"> 1) Check whether the plug is well connected and working well 2) Check whether the switch on the Massage Chair is turned to position "I" |
| During operation, the controller is working but not the Massage Chair | The Massage Chair could be overheated after prolonged use. It will automatically cool down to resume its normal condition. Please turn OFF the power and allow the chair to cool down for 30 to 50 minutes. |
| During operation, both the Controller and the Massage Chair are not working | <ol style="list-style-type: none"> 1) If the set-time in the Timer is complete, please press the timer function again to resume operation. 2) Switch OFF power, and switch ON again to activate the Chair. |

If above solutions are not workable, kindly refer to "product warranty card" and contact Ogawa Service Center.

11 Product Maintenance

Do not use chemical base cleaning detergent to clean the surface of the chair. Use only water.

Do not stack, pull harshly, stab or slit the surface of the massage chair.

Press the controller's buttons with gently during operation, avoids hitting with hard object.

Always pull out the power supply wire plug after using.

Recommend duration of the operation is 30 minutes. It is not advisable to exceed the suggested time frame.

30)AIR INTENSITY

The Intensity Adjustment Button for Air pressure massage, 5 levels of intensity to be adjusted

When Air massage function is engage, every single press on this button will change the massage speed interval in the sequence of "Lv1 > Lv2 > Lv3 > Lv4 > Lv5" in cycle.

31)SPEED

The Speed Adjustment Button for Tapping, Clapping, Swedish and Shiatsu function

When Tapping, Clapping, Swedish or Shiatsu function is engage, every single press on this button will change the massage speed interval in the sequence of "Lv1 > Lv2 > Lv3 > Lv4 > Lv5" in cycle.

32)STRENGTH

The Strength Adjustment Button for Kneading function

When Kneading function is engage, every single press on this button will change the massage speed strength interval in the sequence of "Lv1 > Lv2 > Lv3 > Lv4 > Lv5" in cycle.

Setting Air Squeeze

33)SHOULDER

Shoulder Air Squeeze Massage Control Button - 5 levels of intensity to be adjusted

Press this button once to activate rhythmic air massage at shoulder. Press this button again to stop the function.

34)BACK & SEAT

Back and Seat Air Squeeze Massage Control Button - 5 levels of intensity to be adjusted

Press this button once to activate rhythmic air massage at back and seat. Press this button again to stop the function.

35)HAND REST

Hand Rest Air Squeeze Massage Control Button - 5 levels of intensity to be adjusted

Press this button once to activate rhythmic air pressure massage at Hand Rest. Press this button again to stop the function.


36)FOOT REST

Footrest Air Squeeze Massage Control Button - 5 levels of intensity to be adjusted

Press this button once to activate rhythmic air pressure massage at Footrest. Press this button again to stop the function.

37)FULL AIR

Full Body Air Pressure Massage Control Button - 5

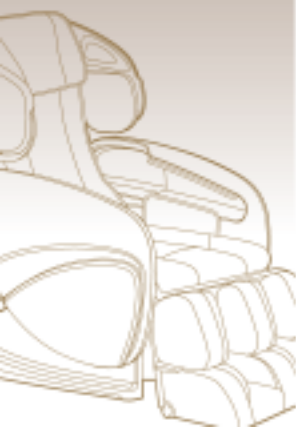Ogawa is grateful
to have received
these awards:

PUTRA
BRAND AWARDS 2019
SILVER
REPUTATION & BRAND VALUE

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